GB Prontosan® – Wound irrigation solution

Instructions for use for cleansing, rinsing and moistening of acute and chronic skin wounds, 1st and 2nd degree burns

1. Introduction: Chronic skin wounds are usually coated with slough, necrotic tissue and/or biofilm. These coatings are extremely difficult to remove and may delay normal wound healing. Therefore it is important to use a wound cleanser with superior cleansing activity like Prontosan® – Wound Irrigation Solution. Proper cleansing and decontamination is also indicated in acute wounds since these are generally contaminated with foreign body, debris and micro-organisms. It is well known that such a contamination can interfere with normal wound healing process, cause a delay in healing and give rise to complications like infection. Prontosan® – Wound Irrigation Solution disperses well and shows an excellent cleansing activity on the tissue surface, both on biofilms and on difficult to access parts of the body and wound.

2. Product profile and areas for use

For cleansing and irrigation plus moistening and decontamination of

- acute wounds: - traumatic wounds (e.g. skin lacerations, bits, cuts or crush wounds)
 - postoperative wounds
- chronic wounds including: pressure sores; venous ulcers; diabetic ulcers
- thermal and chemical wounds: 1st and 2nd degree burns; chemical burns
- post-radiation wounds
- fistulas and abscesses
- entry ports for urological catheters, PEG/PEJ-tubes and drainage tubes
- peristomal skin

For intraoperative cleansing and irrigation of wounds in explorative operations such as severe injury of the extremities.

For irrigation and mechanical cleansing of wounds during instillation therapy (e.g. V.A.C. Instill® Therapy)

For moistening bandages and wound dressings such as compresses, gauze, pads, sponges, hydro fibre, alginates, hydrocolloids and similar and when changing dressings to dissolve encrusted bandages or other encrusted wound dressings.

3. General use: First of all, the wounds should always be irrigated and cleansed with Prontosan® – Wound Irrigation Solution so that coatings that can be readily released have already been removed before further treatment is carried out with Prontosan® – Wound Gel (see separate product information).

Gauzes or pads soaked in Prontosan® – Wound Irrigation Solution can be used for cleansing as required.

Application should be carried out frequently enough for all coatings and necroses to be readily removed and to achieve an optically clean wound. This is a good precondition for normal wound healing and facilitates surgical debridement with optimum visibility.

4. Cleansing the wound site: Thorough cleansing of a large area around the entire wound site with Prontosan® – Wound Irrigation Solution is recommended to minimise the risk of germs being transmitted into the wound. If indicated, a whole body decontamination of the patient using appropriate products should be carried out. Please request our information specific to MRSA-decontamination.

For wounds covering a large area and also wounds difficult to access, bathing the affected area of the body with Prontosan® – Wound Irrigation Solution, for at least 15 minutes, is recommended. The solution should be used in an undiluted form.

Note: Patients with chronic wounds are often sensitive to temperature. Prontosan® – Wound Irrigation Solution can be warmed to body temperature immediately prior to use.

5. Loosen wound dressings and encrusted bandages

Bandages are frequently encrusted and often stick to the wound surface. If these dressings are removed from the wound surface without being soaked (because of time pressure), it can traumatise the wound bed, which in turn delay wound healing. In cases where bandaging is difficult to release, soaking the wound dressings with Prontosan® – Wound Irrigation Solution is recommended until they can be gently removed without traumatising the surface of the wound. If large encrustations are present, the whole area of the body with the dressing should be saturated or bathed in Prontosan® – Wound Irrigation Solution until the dressings can be easily removed.

6. Tissue tolerability and biocompatibility: Non-irritant; well-tolerated; painless; no inhibition of granulation and epithelialisation; evaluated as dermatologically harmless. The excellent tissue compatibility of polihexanide and undecylenamidopropyl betaine has been proven over many years of clinical use, even in patients with long-standing chronic wounds.

7. Side effects: In very rare cases there may be a mild burning sensation after application of Prontosan® – Wound Irrigation Solution but this disappears after a few minutes.

8. Contraindications: Prontosan® – Wound Irrigation Solution should not be used if the patient is known to suffer from allergies or if it is suspected that the patient may be allergic to one of the ingredients. Prontosan® – Wound Irrigation Solution should not be used on the CNS or the meninges, in the middle or inner ear or in the eyes.

Use on hyaline cartilage and in aseptic joint operations is contraindicated. If however Prontosan® – Wound Irrigation Solution does come into contact with aseptic cartilage, it should immediately be irrigated with Ringer solution without lactate or physiological saline.

Do not use in combination with anionic tensides as these may impair preservation.

Prontosan® – Wound Irrigation Solution should also not be combined with other wound cleansing soaps, ointments, oils or enzymes etc. When such substances are removed from the wound, it should be ensured that these have been rinsed out of the wound completely using Prontosan® – Wound irrigation solution.

9. Restrictions of use: Pregnancy and lactation period: There are no hints showing any mutagenic or embryotoxic effects of the ingredients. Since there is no systemic resorption of Polihexanide, a pass over of the ingredients to the breast milk is unlikely. Due to the lack of relevant trials and long-term clinical experience with pregnant and breast feeding women, Prontosan Wound Irrigation Solution should only be used after careful medical consultation in this case. Newborns and infants: Due to insufficient clinical experience, Prontosan – Wound Irrigation Solution should only be used selectively in newborns and infants.

10. General safety instructions: Only for external use on skin wounds. Do not use for infusion or injection! Do not swallow. Use only unobjectionable and undamaged bottle. Keep bottles out of direct sunlight. - Medical device – Keep out of reach of children.

11. Summary / technical information: Prontosan® – Wound Irrigation Solution: Purified water,

0.1 % undecylenamidopropyl betaine, 0.1 % polyaminopropyl biguanide (polihexanide).

Appearance and smell: clear, colourless and virtually odourless, aqueous solution.

Shelf-life: according to the expiry date; storage at room temperature.

Originality: A – sterile, originality seal.

Shelf-life after opening: 8 weeks.

Ampoules: for single use only!

Preservation and shelf-life: Prontosan® – Wound Irrigation Solution is preserved in such a way that the risk of germs being spread to a wider environment during application is considerably reduced.

Therefore, Prontosan® – Wound Irrigation Solution offers an effective germ barrier between the wound and the patient if used according to the instructions. Due to preservation, there is a possible shelf-life of 8 weeks after opening the bottle is the bottle is closed immediately after use and the top of the bottle is protected from contamination or direct contact. Bottles that have come into contact with the wound must be thrown away after use. Ampoules: for single use only!

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